**CLEAR-ALIGNER®**

# Newsletter 7

Casos clínicos

*Case Reports*

CLEAR-ALIGNER®

---

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*All cases were carried out by Dr. Pablo Echarri. All the laboratory work was carried out by Ladent laboratory, and the Scheu Dental Technology materials were used.*

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©2013 Centro de Ortodoncia y ATM, Ladent, SL  
All rights reserved

*This book or any part thereof may not be reproduced, stored in retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying or otherwise, without prior written permission of the publishers.*

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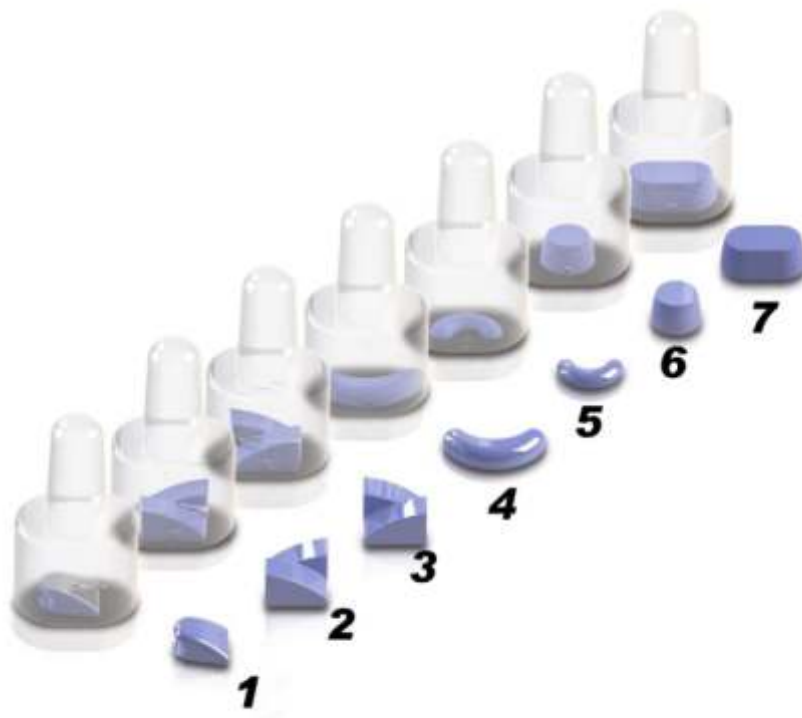
1. MFM Multifunctional Forced Movements
2. MDI Mesial/Distal Inclination
3. DMI Distal/Mesial Inclination
4. FALA Force Application Labial
5. FALI Force Application Lingual
6. BUB Build-Up Bicuspid
7. BUM Build-Up Molar

*The CA Power Grips were designed by Dr. Pablo Echarri, developed by Scheu-Dental, and tested by Ladent, SL and Echarri Clinic. The CA Power Grips consist of seven molds which allow easy attachment of composite retentions to the teeth, which facilitates certain dental movements with the CA Clear Aligner technique. The seven CA Power Grips are:*

1. MFM Multifunctional Forced Movements
2. MDI Mesial/Distal Inclination
3. DMI Distal/Mesial Inclination
4. FALA Force Application Labial
5. FALI Force Application Lingual
6. BUB Build-Up Bicuspid
7. BUM Build-Up Molar

CLEAR-ALIGNER®

## CA Power Grip N° 1: Multifunctional Forced Movement

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*In this Newsletter, we are going to talk about the CA Power Grip 1: MFM, which is used for extrusion, mesial rotation and distal rotation.*

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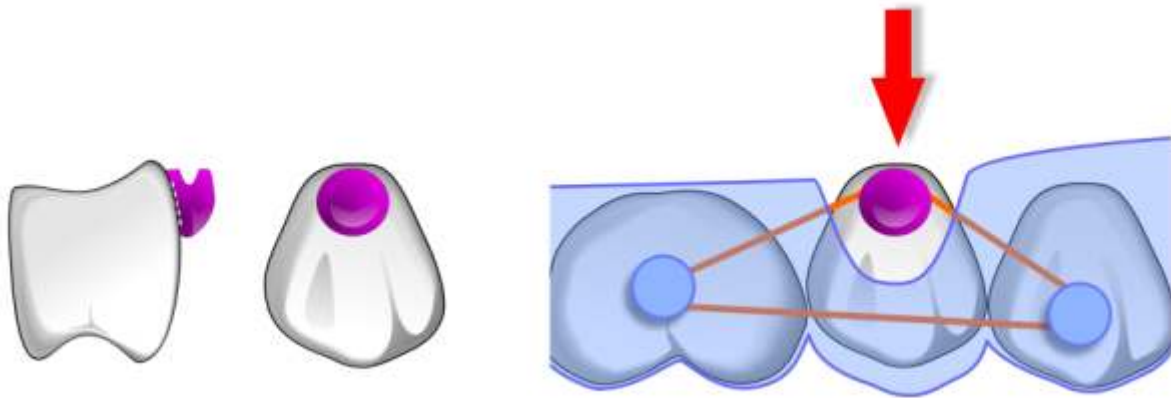
*CA Power Grip 1 MFM for extrusion is bonded on the tooth before taking the impressions. The slot should be positioned in gingival side. Laboratory prepares first set-ups with 0.5 mm of extrusion movement, and then other steps with 1 mm of extrusion movement. Soft and Medium CA for each step are prepared, too. When a tooth reaches the definite position, Clear Aligner Forced Extrusion is also prepared, with two buttons and a cut at the level of the Power Grip. One week, Soft or Medium aligner of each step should be used whenever Clear Aligner FE cannot be used, and CA FE is used while sleeping and at home with elastics of 1/8" and 4.5 oz. The Clear Aligner Forced Extrusion is used from the very beginning of the treatment.*

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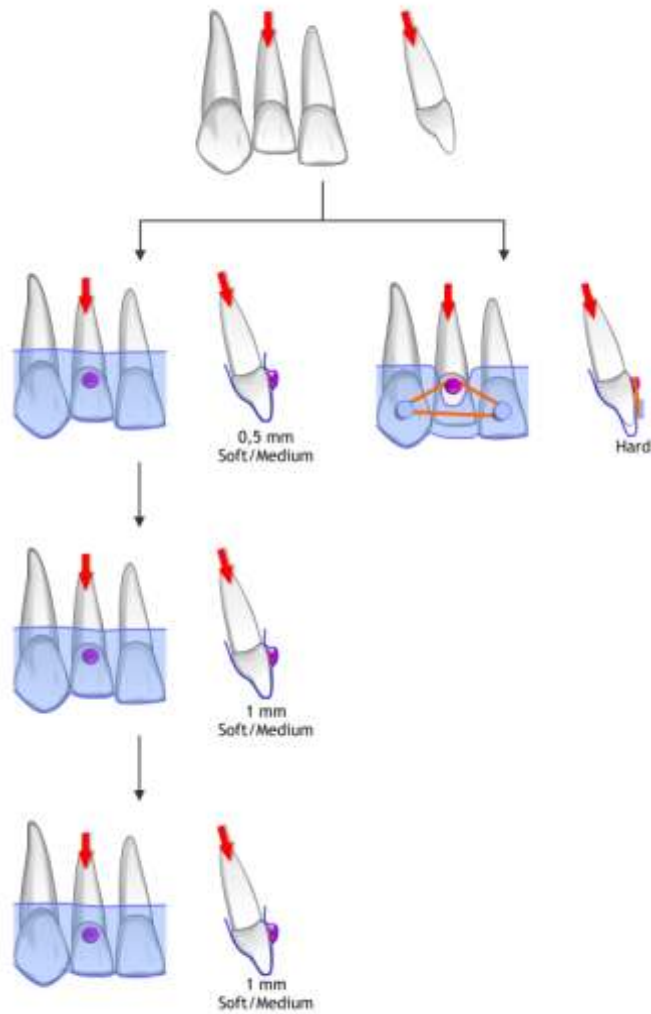
*Bonded button. Scheme of CA Power Grip 1 MFM for extrusion.*

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
*Scheme of the use of CA Power Grip 1 MFM for extrusion.*

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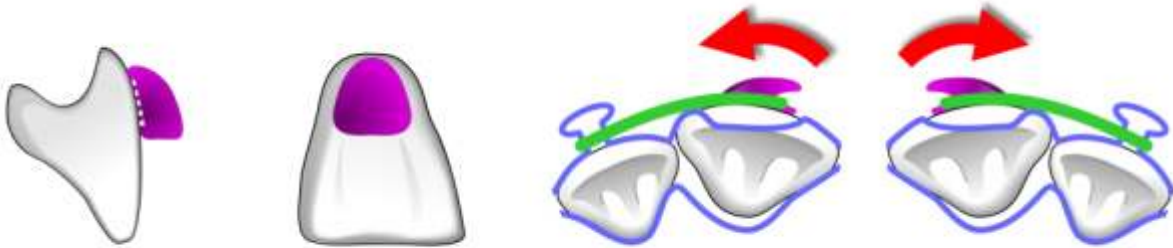
*CA Power Grip 1 MFM can also be used to correct rotations towards mesial or towards distal. In such case, the Power Grip is bonded with the slot towards the direction opposite to the desired movement, and the procedure is the same as for Clear Aligner Forced Extrusion.*

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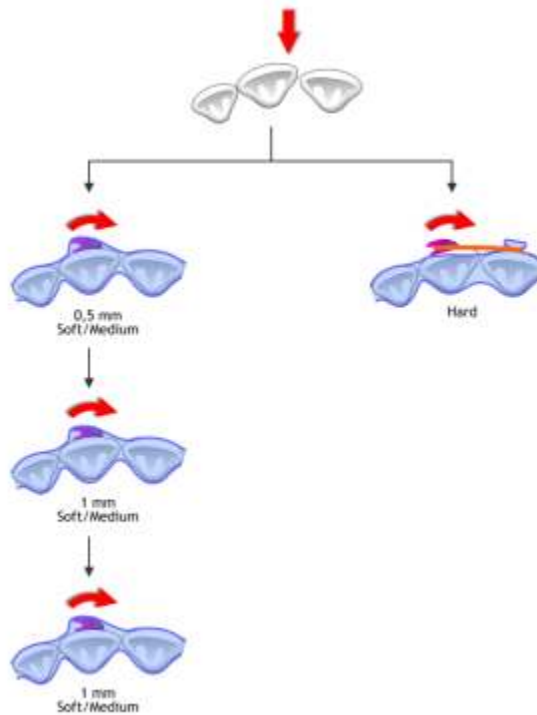*Scheme of the use of CA Power Grip 1 MFM for mesial or distal rotation.*

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*Scheme of the use of CA Power Grip 1 MFM for rotation. Power Grip is bonded and progressive set-ups are prepared, with 0.5 mm of movement in the first step, and 1 mm of the movement in the following steps. The patient uses progressive Clear Aligner Soft ( 1 week) and Medium (1 week) during the day, and at night and at home the patient uses Clear Aligner with elastics from the very beginning of the*

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## CASE 01575

*A patient with a distally rotated upper right lateral incisor.*

*Initial photographs.*

*First, one CA Clear Aligner step is performed to carry out the expansion, forming in this way the necessary space for correction of the lateral incisor rotation.*

*At the following appointment, the CA Power Grip 1 (MFM) was bonded to the upper right lateral incisor.*

- 1. Clean the tooth with a prophylaxis paste.*
- 2. Isolate the site with buccal separators, cotton balls, and aspirate.*

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7. Apply the Primer over the tooth with a brush.

8. Rellenar la forma del CA Power Grip N° 1 MFM con composite\*.

8. Fill the mold of the CA Power Grip 1 MFM with composite.\*

\*Recomendamos: Light Bond de la casa Smile Dental, Tetric Eco Ceram de Ivoclar Vivadent, y Tetric Evo Flow de Ivoclar Vivadent.

\*We recommend: Light Bond from Smile Dental, Tetric Eco Ceram from Ivoclar Vivadent, and Tetric Evo Flow from Ivoclar Vivadent.

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*9. Apply the mold to the tooth, eliminate the excess of the composite and light cure with the light curing lamp for 30 sec.*

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*10. Remove the CA Power Grip 1 form to finish the procedure.*

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*In continuation, the alginate impressions are taken, and CA Clear Aligner used by patient should be cut so the patient can continue using it until the laboratory fabricates the new step.*

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### *Laboratory procedure*

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*The CA Power Grip 1 can be seen on the model.*

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*In this case, the correction could be completed with only one CA Clear Aligner step. The set/up was carried out.*

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*CA SMART was used to check if the movement is correct (1 mm of mesial contraction of the upper right lateral incisor).*

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*Blokkout resin was used for fixation.*

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*The CA Clear Aligner SOFT and MEDIUM were fabricated, covering the CA Power Grip.*

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*CA Clear Aligner HARD Forced Rotation was carried out, it was cut at the level of CA Power Grip, and it has a button.*

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*The patient will use the CA Clear Aligner SOFT for one week, the MEDIUM for two weeks, when he is at work.*

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*The patient will use CA Clear Aligner HARD with an elastic of 1/8" and 4,5 oz at home and while sleeping.*

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*The rotation of this tooth was corrected in five weeks.*